



Outsmart Bronchiectasis

Managing Symptoms with
The SmartVest[®] SQL[®]



smartvest[®]

AIRWAY CLEARANCE SYSTEM

BY ELECTROMED, INC.





What is Bronchiectasis?

Bronchiectasis (brong-kee-EK-tuh-sis)

is an irreversible, chronic condition where airways in the lungs (bronchi) become damaged and abnormally widened from recurring inflammation or infection, preventing your lungs from properly functioning.

Often, bronchiectasis is caused by an underlying condition that may include:

- Respiratory infections
- Immunodeficiency disorders
- Allergic reaction to the fungus *Aspergillus* (ABPA)
- Genetic disorders

*Untreated,
bronchiectasis
may limit your lifestyle*

➤ What are the Symptoms?

Bronchiectasis affects everyone differently. You will probably experience “good” days and “flare ups” known as exacerbations, or a temporary worsening of symptoms.


Common symptoms include:

- Chronic cough that produces mucus
- Recurrent respiratory infections
- Breathlessness and wheezing
- General fatigue

➤ How Does Bronchiectasis Affect Your Lungs?

With bronchiectasis, your airways slowly lose their ability to clear out mucus, which creates an environment vulnerable to infection.

Each recurring infection causes more damage to your airways, which over time impairs their ability to move air in and out.

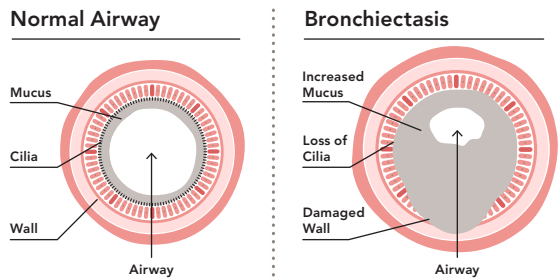

➤ How Do You Manage Bronchiectasis?

Clearing excess mucus from your lungs with a combination of medication and airway clearance techniques is one of the most common ways to improve symptoms and reduce complications of bronchiectasis.

For some patients, these methods aren't enough to provide relief. That's when the SmartVest SQL Airway Clearance System can be an effective, convenient, and comfortable solution to managing bronchiectasis.

Smart Solution: SmartVest SQL

Outsmart Bronchiectasis

The SmartVest SQL delivers rapidly repeating pulses of air that squeeze and release your upper body. These gentle, yet powerful “mini coughs” loosen, thin, and propel mucus towards major airways in your lungs, where it can be more readily expectorated.



Smart Results

The SmartVest SQL Airway Clearance System uses high frequency chest wall oscillation, also known as “HFCWO,” to help mobilize mucus out of your lungs.

You can reduce the need for repeat antibiotic treatments, doctor visits, and hospitalizations by consistently using HFCWO therapy.

30
minute
treatments

2
times
per day

Effective: Reduces your symptoms and improves quality of life, when regularly used.

Convenient: Lightweight and quiet, so you easily integrate therapy into your routine.

Easy to Use: Velcro-like closures, one-touch start, and simple interface make treatment easy.

Comfortable: Superior garment pressure relief allows you to breathe easily during treatment.

Manage bronchiectasis, improve lung function, and live a more active life with the SmartVest SQL Airway Clearance System.

Visit www.smartvest.com or contact Electromed at 800.462.1045.

Customer Care

As a part of our commitment to customer care, our dedicated team will help you manage the reimbursement process and provide you with unrivaled support.

In-Home Support: A licensed medical professional will come to your home and demonstrate proper use for maximum benefit.

Patient Services Team: Our respiratory therapists are available by phone to answer technical and support questions 365 days a year.

Reimbursement Specialists: We coordinate all documentation and provider communications to make the process as easy as possible for you.



The SmartVest SQL is effective, convenient, easy to use, and comfortable.

Reimbursement Support

With proper documentation, the SmartVest SQL Airway Clearance System is reimbursable by private insurance, Medicare, state medical assistance, a combination of these three, or the U.S. Department of Veterans Affairs.

Our team of reimbursement specialists is available to assist with the documentation and reimbursement process.


To qualify for reimbursement, you must have documented:



Daily productive cough for at least six months.


Frequent exacerbations requiring antibiotic therapy (i.e. more than twice annually).

AND

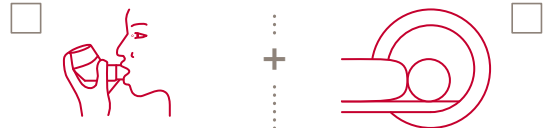

Failure of standard treatments, such as OPEP, to mobilize retained secretions.

HRCT (High-Resolution Computed Tomography) scan confirming bronchiectasis diagnosis.

 **Manage bronchiectasis, improve lung function, and live a more active life with the SmartVest SQL Airway Clearance System.**

Visit www.smartvest.com or contact Electromed at 800.462.1045.



SmartVest SQL

Manage bronchiectasis, improve lung function,
and live a more active life.

Visit www.smartvest.com to learn more.

ELECTROMED, INC.

Making life's important moments possible —
one breath at a time.®

Corporate Headquarters

500 Sixth Avenue NW, New Prague, MN 56071

Phone: 800.462.1045 or 952.758.9299 · Fax: 866.758.5077
info@electromed.com · www.smartvest.com

Engineered and Built in Minnesota

Electromed, Inc., is committed to a strong and continuing program
of innovation for the benefit of patients and their caregivers.



Accredited by The Joint Commission

"SmartVest," "SQL," and "Making life's important moments possible—
one breath at a time" are registered trademarks of Electromed, Inc.